

Combination heat and cold therapy for enhanced flexibility and performance.

put a wrap on pain



VITALWEAR

VITALWRAP®
EQUINE SYSTEM

A breakthrough in injury recovery and prevention. An effective, affordable and veterinarian-approved treatment that's easy enough for everyday use.

The VitalWrap Equine System is an effective and efficient combination of heating, cooling and compression therapy designed specifically for your Equine Athlete. Used before or after workout, the VitalWrap Equine System prepares horses for the demands of a strenuous workout or competition, and can also be used to treat injuries with direct ice or heat treatment. Delivered through specially designed wraps that fit easily onto a horse's body, the VitalWrap Equine System eases pain and accelerates the healing of damaged tissue. The result is a reduced risk of strains and better performance overall.

SIMPLE

Designed for easy, everyday use

PRECISE


Control temperature with one dial, 40° - 105° F

FLEXIBLE

Hot, cold, contrast and compression therapies

IDEAL FOR

Pre-workout and pre-competition;
post-workout and post-competition; warm up;
cool down; flexibility treatment

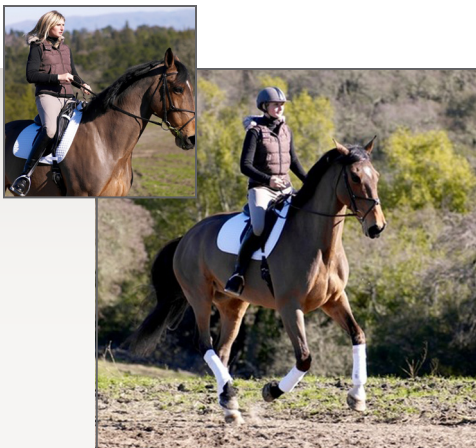

Wide-ranging applications — One versatile solution

Bowed tendon • Chronic lameness • Fetlock lameness • Preventative maintenance • Hock injury
Post-op recovery • Suspensory demitis • Tendon sheath effusion • Many other conditions



VITALWEAR



put a wrap on pain

VitalWrap Equine Bundles

00810 VitalWrap Equine System for Localized Leg Treatment (includes: VitalWrap Equine System (120 V_{ac}), right & left front leg wraps, right & left rear leg wraps, right & left hock wraps, carrying bag, single & Y-splitter tubing set and other accessories)

00820 VitalWrap Equine System for Full Leg Area Treatment (includes: VitalWrap Equine System (120 V_{ac}), right & left full front leg wraps, right & left full rear leg wraps, carrying bag, single & Y-splitter tubing set and other accessories)

00895 VitalWrap Equine System for Spine and Upper Leg Treatment (includes: VitalWrap Equine System (120 V_{ac}), poll wrap, back wrap, sacroiliac wrap, stifle wrap, carrying bag, single & Y-splitter tubing set and other accessories)

Systems

order no.

00221 VitalWrap System 120 V_{ac}, 60 Hz

Wraps

00801 Right Front Leg

00802 Left Front Leg

00803 Right Rear Leg

00804 Left Rear Leg

00805 Right Hock

00806 Left Hock

01032 Right Full Front Leg

01033 Left Full Front Leg

01024 Right Full Rear Leg

01025 Left Full Rear Leg

01034 Poll

01035 Stifle

01036 Back

01037 Sacroiliac

00001 6" Universal Wrap

00118 8" Universal Wrap

Accessories

00857 Equine Carrying Bag

00948 Tote Bag

00949 Long Elastic X 1

00950 Short Elastic X 2

00994 Y-Splitter tube set

00034 Single tubing set

00262 Power cord

SYSTEM SPECIFICATIONS:

PHYSICAL

size (approximate)

8" diameter x 13.5" height
(20.3 cm x 34.3 cm)

weight (dry)

5 lbs (2.27 kg)

fluid tether

8' (2.44 m)

CONTROL SYSTEM

type

manual user control

THERMAL SYSTEM

range

40° - 105° F (4.4° - 40.6° C)

contrast therapy transition time

1 min typical from extremes in temperature range

operating period

2-6 hours typical continuous use

thermal cutout

116.6° F (47° C)

CIRCULATING SYSTEM

reservoir capacity

1.1 gal (4.2 L)

reservoir fluid

ice water

flow rate thru VW

5.5 gph (21 Lph)

maximum pressure

18 psi

ELECTRICAL SYSTEM:

voltage

100, 120 V_{ac}, 50/60 Hz or
220-240 V_{ac}, 50 Hz

power

300 W

internally fused

system and pump

leakage current

under 300 µA

OPERATING ENVIRONMENT:

atmospheric pressure

525 - 795 mmHg

humidity

30% - 70% relative non-condensing

temperature

10° C - 40° C

TRANSPORT & STORAGE ENVIRONMENT:

atmospheric pressure

179 TO 760 mmHg

humidity

10% - 95% relative non-condensing

temperature

0° C - 50° C

VitalWear, Inc.

384 Oyster Point Blvd., Suite 16
South San Francisco, CA 94080-1968
Tel: 1-800-553-4081
www.vitalwear-equine.com